

Maxi Ratcheting Screwdriver System



Surgical Technique Guide



MKS 1029		Maxi Ratcheting Screwdriver System		
No.	Part Number	Description	Image	QTY
1	MDS1123-M01	2.0mm Hex Driver (SOLID)		1
2	MDS1124-M01	2.5mm Hex Driver (SOLID)		1
3	MDS1125-M01	3.5mm Hex Driver (SOLID)		1
4	MDS1126-M01	4.0mm Hex Driver (SOLID)		1
5	MDS1127-M01	5/32" Hex Driver (SOLID)		1
6	MDS1128-M01	3/16" Hex Driver (SOLID)		1
7	MDS1129-M01	4.5mm Hex Driver (SOLID)		1
8	MDS1130-M01	5.0mm Hex Driver (SOLID)		1
9	MDS1131-M01	T-8 Torx Driver (SOLID)		1
10	MDS1132-M01	T-15 Torx Driver (SOLID)		1
11	MDS1133-M01	T-20 Torx Driver (SOLID)		1
12	MDS1134-M01	T-25 Torx Driver (SOLID)		1
13	MDS1135-M01	T-30 Torx Driver (SOLID)		1
14	MDS1136-M01	Cruciform Driver (SOLID)		1
15	MDS1137-M01	Flat Tip Driver (SOLID)		1
16	MDS1138-M01	Phillips Driver (SOLID)		1
1	MDS1139-M01	2.0mm Hex Driver (FLEXIBLE)		1
2	MDS1140-M01	2.5mm Hex Driver (FLEXIBLE)		1
3	MDS1141-M01	3.5mm Hex Driver (FLEXIBLE)		1
4	MDS1142-M01	4.0mm Hex Driver (FLEXIBLE)		1
5	MDS1143-M01	5/32" Hex Driver (FLEXIBLE)		1
6	MDS1144-M01	3/16" Hex Driver (FLEXIBLE)		1
7	MDS1145-M01	4.5mm Hex Driver (FLEXIBLE)		1
8	MDS1146-M01	5.0mm Hex Driver (FLEXIBLE)		1
9	MDS1147-M01	T-8 Torx Driver (FLEXIBLE)		1
10	MDS1148-M01	T-15 Torx Driver (FLEXIBLE)		1
11	MDS1149-M01	T-20 Torx Driver (FLEXIBLE)		1
12	MDS1150-M01	T-25 Torx Driver (FLEXIBLE)		1
13	MDS1151-M01	T-30 Torx Driver (FLEXIBLE)		1
14	MDS1152-M01	Cruciform Driver (FLEXIBLE)		1
15	MDS1153-M01	Flat Tip Driver (FLEXIBLE)		1
16	MDS1154-M01	Phillips Driver (FLEXIBLE)		1
—	MHRS0311	Cannulated Blue Silicone Ratcheting Screwdriver		1
—	CA174-M01	Case		1

Contact us at: **888-4shukla (888-474-8552)**
 or **customer.service@ShuklaMedical.com**
 See our website at: **www.ShuklaMedical.com**

Maxi Ratcheting Screwdriver System

<p>Step 1 - Determine the drive configuration of the bone screw.</p>	<p>CRUCIFORM PHILLIPS TORX FLAT HEX</p> 
<p>Step 2 - Based on accessibility of the working site choose either the straight or flexible corresponding configuration screwdriver.</p>	
<p>Step 3 - Insert the 1/4" square end of the bone screwdriver into the blue silicon ratcheting handle. The driver shaft should click into place.</p>	
<p>Step 4 - Rotate the cap to the left for removing screws, the center position for locking the handle into place and to the right for installing screws.</p>	 <p>REVERSE LOCKED FORWARD</p>
<p>Step 5- The ratcheting screwdriver is now ready to remove or install bone screws.</p>	

				
<p>EMERGO GROUP Molenstraat 15 2513 BH, The Hague The Netherlands Tel: +31 (0)70 345 8570 Fax: +31 (0)70 346 7299</p>	<p>MKS1029</p>	<p>Shukla Medical 151 Old New Brunswick Road Piscataway, NJ 08854 U.S.A. 1(888)-4SHUKLA www.shuklamedical.com</p>	<p>CONSULT INSTRUCTIONS FOR USE</p>	<p>NON STERILE PRODUCT</p>